
iCart Prophy

Operation Manual

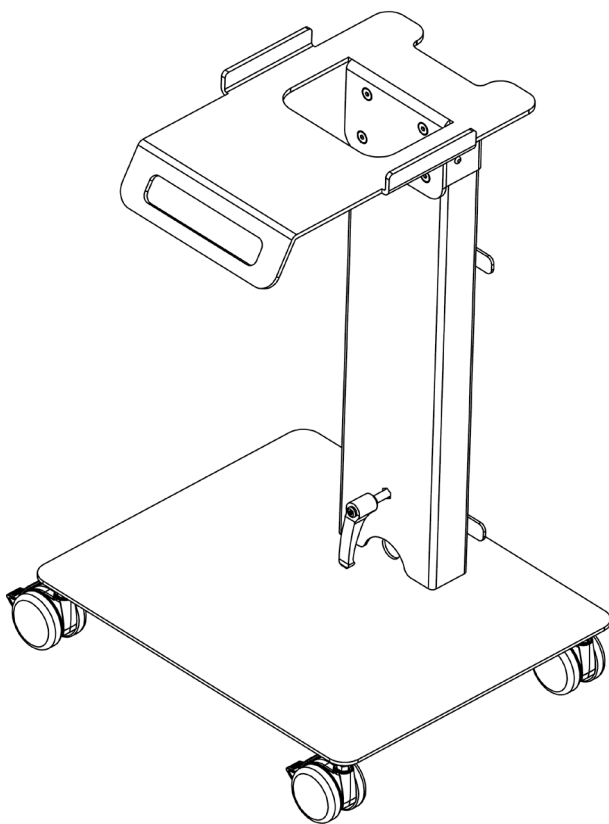
1 – Introduction

iCart Prophy is a dental cart specifically designed to support NSK's equipment such as Varios Combi Pro. It includes bracket on the rear of the column to rest the power supply cord, and to avoid any unexpected movement during use, the front wheels are fitted with brakes.

Upon receipt, you will find

- Basis equipped with wheels (1)
- Adjustable column, equipped with mounting screws (2)
- Shelf for medical device (3)
- Set of screws for shelf and hex key for assembly (4)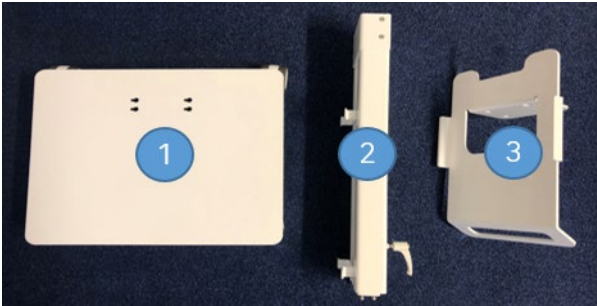

2 - Operating manual


2.1 Assembly instruction

In order to assemble the cart, proceed as follows:

- Insert the screws located on bottom of the column through the corresponding opening of the base plate
- Slide the column as illustrated below and tighten the screws to secure it
- Attach the shelf to the upper part of the mast using the countersunk screws

Step 1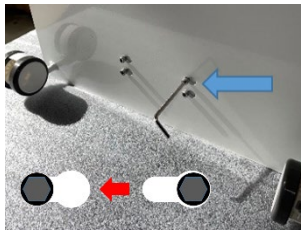

Step 2


Step 3

Be careful not to fix the shelf upside down.

2.2 Description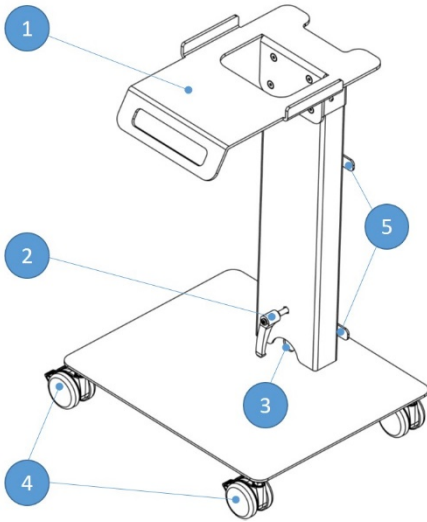

- 1- Shelf for NSK device
- 2- Thumb screw for height adjustment
- 3- Hole for foot control cord
- 4- Front wheels with brake
- 5- Reel for power cord

2-3 Installation

After assembly:

- Unscrew the thumb screw (2)
- Adjust the height
- Tighten firmly the thumb screw

Once these steps are achieved

- Install the equipment onto the shelf (1)
- Place the footswitch on the base plate
- Route the footswitch cord through the corresponding hole (3)
- Connect the footswitch cord to the back of the device

	<p>If adjusting the height, firmly hold the shelf when unscrewing the thumb screw. Otherwise, they can move down sharply and injure your hands.</p>
--	---

2-4 Characteristics

<p>Weight : 21 Kg Size H/W/D: 660x450x380mm</p>	<div style="display: flex; align-items: center;"> <p>Do not waste in domestic bin. This symbol indicates that this device should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.</p> </div>
--	---

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NSK Europe GmbH